
Quick start guide CCS3A-2217-EU-TL Care Call SignWave Interconnect

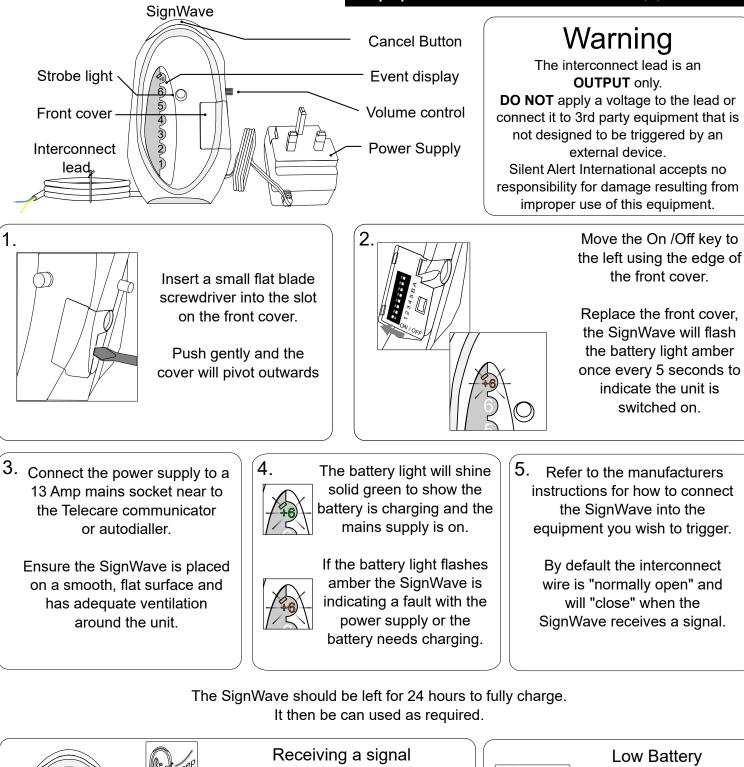

Receiv When a signal is will flash, a sour one of the zone This sequence pressing the ca of If it is not canc the sequence w

543

When a signal is received the strobe will flash, a sound will be emitted and one of the zone lights will illuminate. This sequence can be cancelled by pressing the cancel button at the top of the unit.

If it is not cancelled with the button the sequence will continue for 20 - 30 seconds then cancel automatically. Turn the control clockwise to increase volume or anticlockwise to decrease.



Any questions? Call us on +353 (0)1 247 9007

Low Battery When the SignWave has a low battery the battery light will flash orange.

The unit will beep, the strobe will flash and the battery symbol will flash 4 times. This sequence will repeat every 15 minutes until the SignWave is fully charged.

Advanced settings CCS3A-2217-EU-TL


Care Call SignWave Interconnect



Any questions? Call us on +353 (0)1 247 9007

Strobe and sounder options.


It is possible to change the pitch of the sound produced and deactivate both the strobe and sounder.

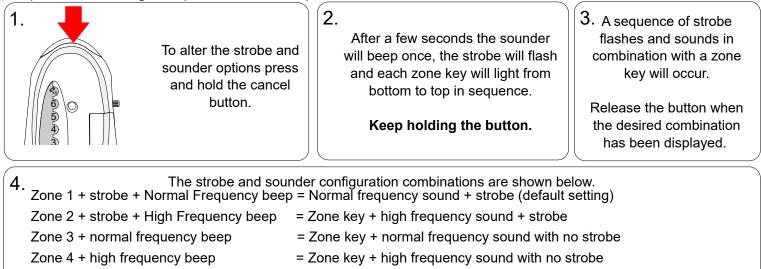

Zone 5 + strobe + no beep = Zone key, no sound + strobe

Increasing system range


It is possible to increase the range of the system.

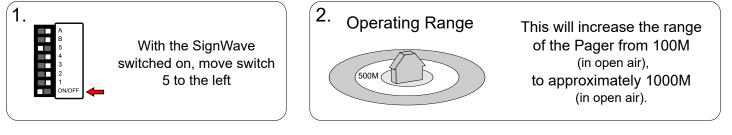

Changing the system channel code

NOTE. In most cases it is not necessary to change the system code. However, when one or more systems are in close proximity, system codes can be used to avoid interference from other Care Call systems using up to a maximum of 16 channels.

The SignWave is supplied with no 1 2 3 4 5 6 7 8 system code switches set (system code 1). For reference this is the factory setting should you need to re-set the unit. System codes can be set using key switches 1 - 4. The diagram to the right shows the 16 9 10 11 12 13 14 15 16 possible combinations. Be sure that the same system code is set on all monitors to be used in the system. NOTE. Care Call receivers have an ON/OFF switch. Care Call monitors do not. Here we have a Care Call SignWave and Mini Monitor set to INCORRECT CORRECT system channel 2. Ensure when setting system channel keys you remember to ignore the SignWave ON/OFF switch or the equipment will be on different system channels The diagram to the right shows an incorrect attempt to set the system to channel code 2 (shown as an example to avoid possible confusion).

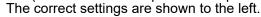

Interconnect settings CCS3A-2217-EU-TL


Care Call SignWave Interconnect



Changing the interconnect output

Any questions? Call us on +353 (0)1 247 9007

It is possible to to change the default setting of normally open to normally closed. NOTE: Only use this feature for equipment that require a "normally closed" input.

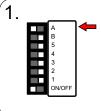

With the SignWave switched on, move switch A to the left This will close the relay contacts. The contacts will "open" when a signal is received.


Transmitter Settings

The Interconnect SignWave will receive any Care Call signal sent from a transmitter on the same system code.

2.


HOWEVER: to trigger an interconnected device the transmitter must be configured to a special zone 13

CCF3A-2219-EU, CCF3A-2219-EU-GFOB, CCS-CO-DETECTOR, CCS-SMOKE-ION, CCS-SMOKE OPTO, CCS-HEAT-DETECTOR

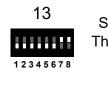


Switch keys 7 & 8 on. The transmitter will now transmit zone 13.

CCM4A-2214-EU, CCC4A-2212-EU, CCD4A-2220-EU

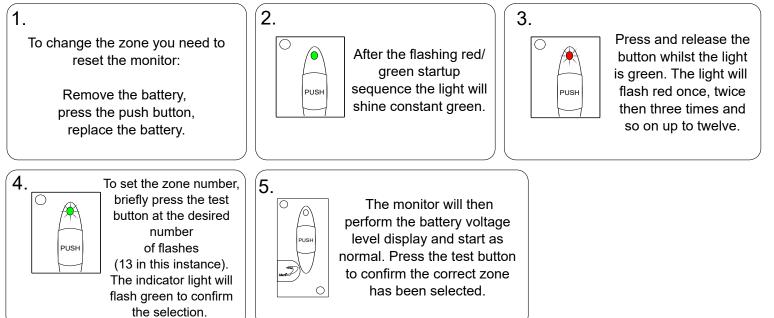

CCM3A-2216-EU, CCM3A-2216-EU-PIR

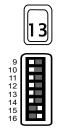

Switch keys 15 & 16 on. The transmitter will now transmit zone 13.