

## FSCCH-2226-IN Fire Safe Charger

Part of the Silent Alert Fire Safe Paging System for deaf and deaf-blind people



Fire Safe Charger



Pager docked in  
charging slot



Power Supply



Power  
Supply  
Plug

Pillow Pad /  
Strobe Light  
Plug



Pillow Pad (optional)

### Getting Started

1. Locate the Fire Safe Charger at reception or if the system is to be used overnight place it on a bedside table and connect the power supply and pillow pad.
2. Plug the Power Supply into a 13amp mains socket. Leave the charger switched on at all times as it has an integral emergency back-up battery.
3. Ensure the pager unit is switched on and dock it onto the charging plug ensuring the pager is pushed fully down. If there is sufficient charge in the battery the pillow pad will activate three times to confirm that the pager has been docked correctly. Charge the pager for 24 hours.
4. Install the Fire Safe Interface according to the manual supplied with the unit.
5. To reset the pager and stop the vibration/flashing, press and hold the key that is lit. Each signal will automatically cancel after 20 - 30 seconds except the fire key, which activates for 15 minutes.
6. The battery key on the pager will not flash whilst it is in the charger.
7. The Pager should be left on charge whilst ever it is not in use to ensure the integral battery is maintained.

**DO NOT IMMERSE THE TRICKLE CHARGER IN WATER OR ANY OTHER LIQUID. DO NOT SUBJECT THE UNIT TO EXCESSIVE MECHANICAL SHOCK. THE FIRE SAFE CHARGER SHOULD BE PLACED ON A SMOOTH HARD SURFACE AND AWAY FROM OTHER OBJECTS TO ENSURE CORRECT VENTILATION. DO NOT COVER UNIT OR PLACE ON SOFT OR CARPETED SURFACES.**

The Silent Alert Fire Safe System comprises a Pager Unit, a Fire Safe Charger and a Fire Safe Interface.